



Vitamin B1 and vitamin B6 in human blood

Illustration of analytical performance for vitamin B1 and vitamin B6 in human blood.

The SCIEX Citrine MS/MS system is intended to identify inorganic or organic compounds in human specimens. All laboratory-developed tests must be developed, verified and validated in accordance with applicable laws and regulations prior to their use for clinical diagnostic purposes.

This document describes a test of the analytical performance of the SCIEX Citrine MS/MS system to analyze vitamin B1 and vitamin B6 in human blood matrix.

The analytical performance data presented here is for illustrative purposes only to demonstrate the potential capabilities of the system. Performance in individual laboratories may differ due to a number of factors, including system configuration, laboratory methods, and operator technique. This document does not constitute a warranty of merchantability or fitness for any particular purpose, express or implied, including for the testing of the compounds analyzed in this experiment.

Materials and methods

The Citrine MS/MS system was controlled, and data processed using Analyst MD software, version 1.6.3. Blood calibrators, controls and samples were processed using the following conditions:

Sample preparation: Sample preparation was performed using Diagnostix's vitamin B1 and B6 reagents set (<https://www.diagnostix.com/en/products/vitamin-b1-b6-reagent-set>) according to the manufacturer's specifications. A 50 µL blood sample spiked using the set of calibrators was used for the procedure.

Liquid chromatography conditions: Chromatographic separation was achieved using a Phenomenex Kinetex C18 column. Mobile phases A and B from the reagents set were used. The total run time was 5 minutes at a flow rate of 600 µL/min. The injection volume was 20 µL.

Mass spectrometry conditions: Mass spectrometry analysis was performed using the Citrine Triple Quad MS/MS system, operating in positive electrospray mode. Compound-dependent parameters were optimized by infusion.

Results

Analytical performance statistics including the concentration range evaluated, accuracy and precision (n=4 replicates), as well as signal-to-noise ratio (S/N) and linearity (r²) are shown in Table 1. Chromatograms of the compounds evaluated utilizing the described method are shown in Figure 1. Calibration curves over the defined concentration ranges for each compound are illustrated in Figure 2.

Table 1. Performance statistics for the analysis of vitamin B1 and vitamin B6 in human blood. Measured range (nmol/L), % accuracy, %CV, S/N ratio and linearity for vitamin B1 and B6. Values for the lowest calibrator (39 nmol/L for vitamin B1 and 21 nmol/L for vitamin B6) and over the measured ranges (39-657 nmol/L for vitamin B1 and 21-361 nmol/L for vitamin B6) were used, as appropriate.

Compound	Range [pg/mL]	% Accuracy	%CV	S/N*	Linearity (r ²)
Vitamin B1	39-657	99.58	5.2	190.1:1*	0.9999
Vitamin B6	21-361	100.76	1.8	552.3:1*	0.9997

*S/N ratio calculated using a peak-to-peak algorithm for lowest matrix calibrator measured.

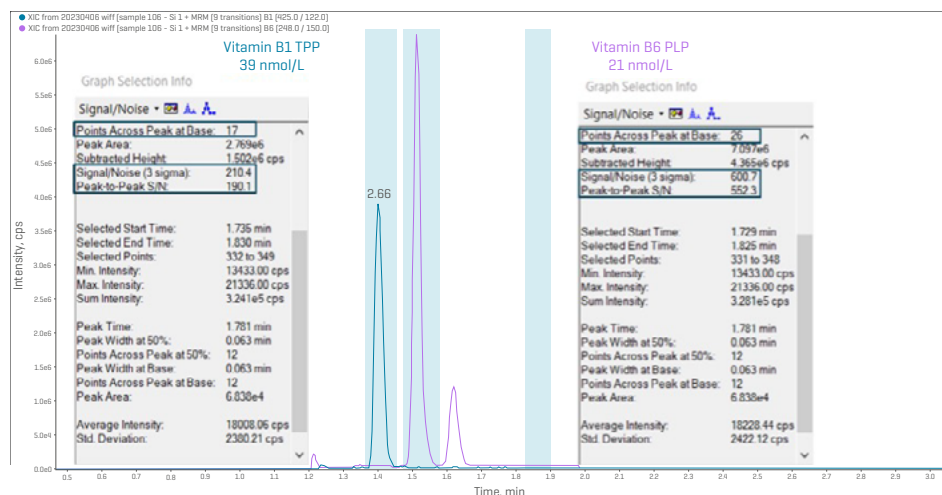


Figure 1. Chromatogram of vitamin B1 (blue) and vitamin B6 (purple) extracted from blood matrix. Chromatogram of calibration standards in matrix for vitamin B1 TPP at 39 nmol/L and vitamin B6 PLP at 21 nmol/L shows a S/N of 190.1:1 for vitamin B1 and 552.3:1 for vitamin B6 based on a peak-to-peak algorithm.



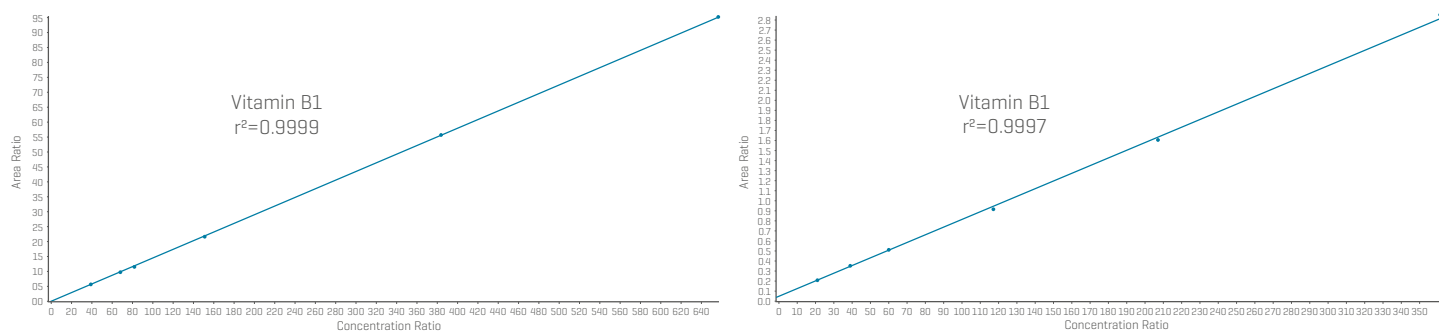


Figure 2. Linear calibration curves for vitamin B1 (left) and vitamin B6 (right) extracted from blood matrix. The calibration curves were run across the following concentration ranges [39-657 nmol/L for vitamin B1 and 21-361 nmol/L for vitamin B6]. The curves were generated using linear regression and 1/x weighting for vitamin B1 and vitamin B6 in blood, resulting in r^2 values of 0.9999 and 0.9995, respectively.

Conclusions

Based on the above performance testing, the following results were obtained:

Sensitivity: Analytical sensitivity was investigated with a series of calibration standards prepared as described and showed a S/N of 190.1:1 for vitamin B1 and 552.3:1 for vitamin B6, at the lowest matrix calibrator measured [39 nmol/L for vitamin B1 and 21 nmol/L for vitamin B6], calculated using a peak-to-peak algorithm.

Assay linearity: Linearity was assessed in matrix over the following concentration ranges: 39-657 nmol/L for vitamin B1 and 21-361 nmol/L for vitamin B6. The r^2 values were 0.9999 and 0.9997, respectively.

Accuracy: At the lowest matrix calibrators measured [39 nmol/L for vitamin B1 and 21 nmol/L for vitamin B6], the % accuracy was 99.58% for vitamin B1 and 100.76% for vitamin B6, determined by 4 replicates in matrix. Data evaluated is based on calculated concentration with internal standard.

Reproducibility: At the lowest measured calibrators [39 nmol/L for vitamin B1 and 21 nmol/L for vitamin B6], the precision [%CV] was 5.2% for vitamin B1 and 1.8% for vitamin B6, determined by 4 replicates in matrix. Data evaluated is based on calculated concentration with internal standard.

In these experiments, the Citrine MS/MS system exhibited the capability to deliver sensitive and reproducible analytical performance for the quantitation of vitamin B1 and vitamin B6 in blood matrix.

Acknowledgements

The data presented here is courtesy of and produced in collaboration with Diagnostix, Appingedam, The Netherlands using their vitamin B1 and B6 reagents set (<https://www.diagnostix.com/en/products/vitamin-b1-b6-reagent-set>). Laboratory work and data processing was carried out at Diagnostix by Judith Boom and Fabian Reijn.